

Enhanced performance

ALLEVYN Adhesive and ALLEVYN Sacrum now handle up three times more fluid through better breathability and rate of absorption.

This optimises the moist wound environment leading to:

- Promotion of faster healing of the wound
- Reduced risk of maceration

Successful healing feels good

New
Nouveau

*This detail aid relates to ALLEVYN Adhesive, ALLEVYN Sacrum, ALLEVYN Plus Adhesive and ALLEVYN Plus Sacrum

The world's No.1 foam dressing...

ALLEVYN[®] – delivers to patients and carers alike

- ✓ Proven clinical efficacy
- ✓ Ensuring moisture balance
- ✓ Patient-friendly comfort and protection
- ✓ Cost-effective quality care

...now with enhanced performance



Introducing ALLEVYN[®] the dressing designed for enhanced performance

Improved
breathability



Faster fluid
uptake




Up to 3 times
more fluid
handling capacity



Delivers the optimal moist
wound environment for the
promotion of faster healing

The dressing designed for enhanced performance

ALLEVYN[◇] – The way it works is why it works

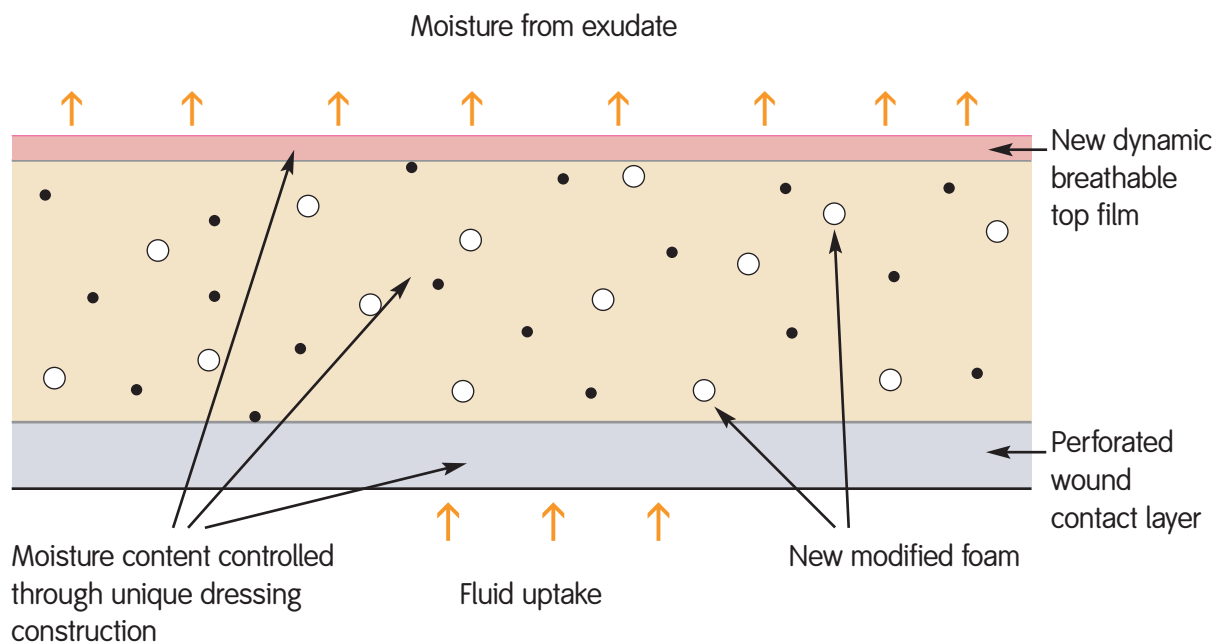


- New top film that becomes more breathable when you need it most*
- Faster fluid uptake due to new modified foam**
- Advanced triple-layered construction
- Facilitates wound healing through dynamic fluid management

*4022/V3/Data on file report/ALLEVYN

**3291/V2/Data on file report/ALLEVYN



ALLEVYN[®] – Manages moisture when it needs to,
where it needs to

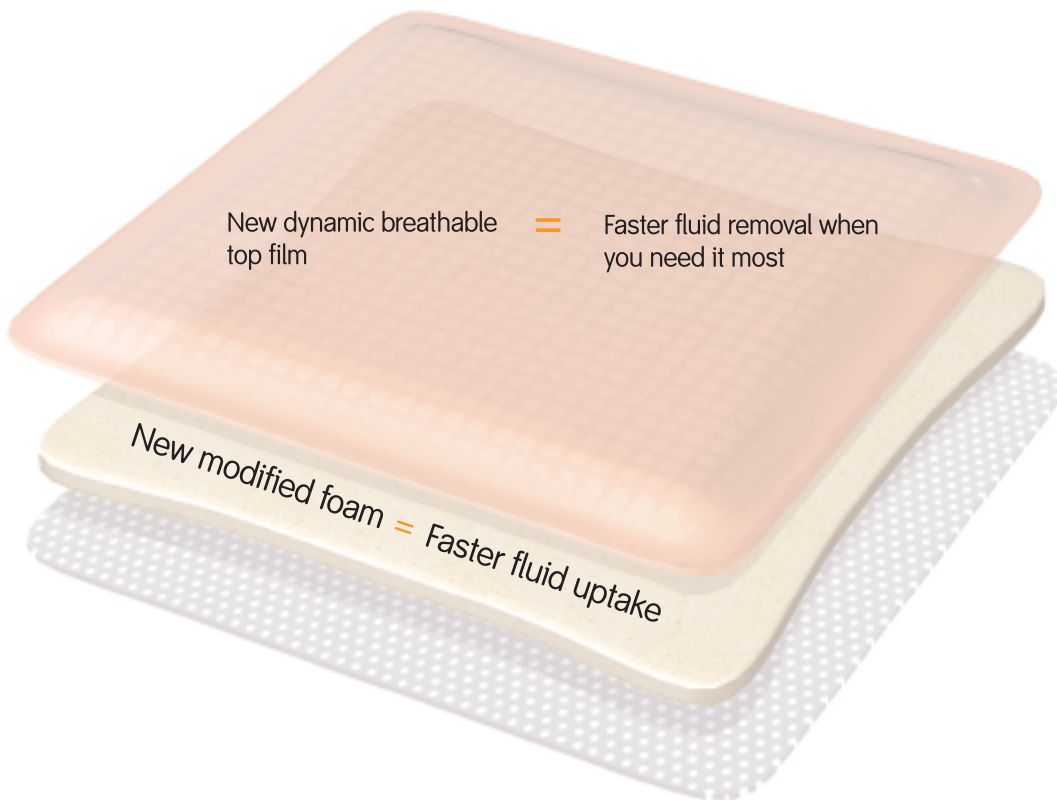

Delivers the optimal moist wound environment for the promotion of faster healing

A new type of dressing designed for optimal fluid management

Consequences of poor fluid management

- Excessive moisture – leads to maceration
- Lack of moisture – drying out of wound



Example of maceration



Example of dry wound


How ALLEVYN helps to facilitate healing?

- ALLEVYN reduces the risk of maceration
- ALLEVYN maintains an optimal moist wound environment



ALLEVYN[®] – Delivering superior fluid management

Fluid handling capacities of a range of adhesive dressings over a 24 period

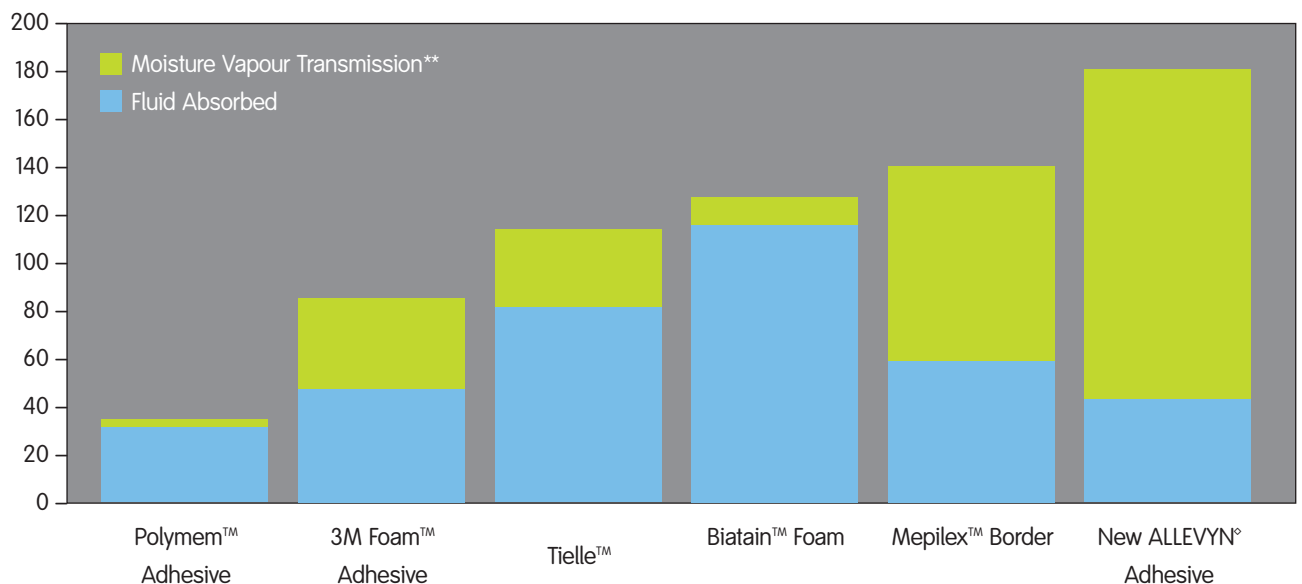

ALLEVYN Adhesive now has the best fluid handling capacity within foam dressings

Reduced dressing changes



Reduced risk of maceration



Increased patient comfort and convenience for you

Reviewing the use of a sacral dressing (ALLEVYN[®] Plus Sacrum)

Aim of the study

The aim of the study was to investigate the potential use and appropriateness of ALLEVYN Plus Sacrum in sacral pressure ulcers, particularly in absorbency and remaining in place during wear time.

The study reviewed 20 patients (all gave full informed consent) with pressure ulcers with each patient being visited weekly over a 6-week period.

Materials and methods

The measurements were both qualitative and quantitative with assessments based on TIME[®] framework and the continuum of wound healing.

The quantitative planimetry measurements were taken and Vistrak (A portable digital device that provides accurate, reproducible data for tracking wound progress) provided the wound surface measurements and photographs confirm the changes in colour and size of the wound.

Qualitative measurements investigated comfort, absorbency, adhesion, ease of removal and durability.

The nurses were asked to comment on ease of application, wear-time, absorbency and ease of removal of trial dressing.

The ALLEVYN Plus Sacrum was left in situ until strike through was apparent and was changed by the patient's primary nurse.


Results



Each of the wounds improved according to the TIME framework
Each wound moved through the wound continuum towards granulation
One wound healed during the evaluation

Three patient case studies

Patient 1

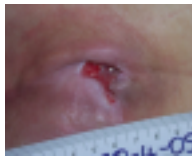

A sacral pressure ulcer grade 3



The same wound after 6 weeks



ALLEVYN Plus Adhesive Sacrum newly applied



ALLEVYN Plus Adhesive Sacrum after 3 days wear

Patient 2



Excoriation and pressure damage grade 2



Dressing still in place after 5 days




Wound healing after 15 days

Patient 3



90 year old lady with a large necrotic pressure ulcer grade 4 (Stirling), caused by sudden immobility

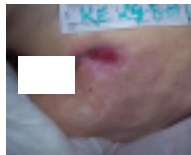

The same wound after 7 weeks of ALLEVYN Plus Adhesive Sacrum



ALLEVYN Plus Adhesive Sacrum applied. This wound is now completely healed

Conclusion

- The dressing was not dislodged from the wound on any occasion
- Extremely effective in absorbing wound fluid
- The dressing was easily applied and easily removed without damaging the friable skin
- Wear time ranged from daily (when faecal incontinence was a problem) to 4/5 days
- There were signs of healing in all wounds in the case studies with complete healing occurring in one patient

Moving wound care forward

The responsive dressing that adapts to its environment

Optimising moisture control for patient comfort and convenience

A dressing that helps your patients by protecting their wound

Delivering cost effective care



The ALLEVYN® products that now have enhanced performance are:

- ALLEVYN Adhesive
- ALLEVYN Sacrum
- ALLEVYN Plus Adhesive
- ALLEVYN Plus Sacrum

Ordering information



NEW ALLEVYN Adhesive



Code	Size	Carton
6600043	7.5cm x 7.5cm	10
6600044	12.5cm x 12.5cm	10
6600744	12.5cm x 22.5cm	10
6600045	17.5cm x 17.5cm	10
6600046	22.5cm x 22.5cm	10
6600599	10cm x 10cm	10

NEW ALLEVYN Sacrum



Code	Size	Carton
66000451	22cm x 22cm	10
66000700	17cm x 17cm	10

ALLEVYN Plus Adhesive



Code	Size	Carton
66000805	12.5cm x 12.5cm	10
66000806	17.5cm x 17.7cm	10
66000807	12.5cm x 22.5cm	10

ALLEVYN Plus Sacrum



Code	Size	Carton
66000857	22cm x 22cm	10
66000856	17cm x 17cm	10

ALLEVYN Lite



Code	Size	Carton
66000690	5cm x 5cm	20
66000691	10cm x 10cm	20
66000692	10cm x 20cm	20
66000698	15cm x 20cm	20

ALLEVYN Tracheostomy



Code	Size	Carton
66007640	9cm x 9cm	10

ALLEVYN Heel



Code	Size	Carton
66007630	10.5cm x 13.5cm	5

ALLEVYN Non-adhesive



Code	Size	Carton
66007643	5cm x 5cm	10
66007637	10cm x 10cm	10
66007335	10cm x 20cm	10
66000033	15cm x 15cm	10
66007638	20cm x 20cm	10
66000663	40cm x 70cm	2

ALLEVYN Compression



Code	Size	Carton
66047581	5cm x 6cm	10
66047583	10cm x 10cm	5
66047584	15cm x 20cm	3
66045125	15cm x 15cm	3

ALLEVYN Cavity



Code	Size	Carton
66007326	Small circular	10
66007327	Large circular	5
66007328	Small tubular	10
66007329	Large tubular	5

ALLEVYN Plus Cavity



Code	Size	Carton
66047571	5cm x 6cm	10
66047573	10cm x 10cm	5
66047574	15cm x 20cm	3

ALLEVYN Thin



Code	Size	Carton
66047576	5cm x 6cm	10
66047578	10cm x 10cm	5
66047579	15cm x 20cm	3
66045124	15cm x 15cm	3

Wound Management
Smith & Nephew
Medical Ltd
101 Hessle Road
Hull HU3 2BN

www.smith-nephew.com/wound
www.allevyn.com

T 44 (0) 1482 225 181
F 44 (0) 1482 328 326