

ShotBlocker is easy to use and works with all types of safety syringes and needles.



1. Select the injection site and prep the skin as usual.
2. Hold ShotBlocker® so that the blunt contact points touch the skin at the injection site.
3. Press ShotBlocker® firmly against the skin. **DO NOT MOVE OR REMOVE SHOTBLOCKER UNTIL THE INJECTION HAS BEEN COMPLETED.**
4. Immediately administer the injection in the usual manner through or near the central opening of ShotBlocker®. For subcutaneous injections, angle the needle as needed to give the injection. **IF MORE THAN 20 SECONDS ELAPSE BETWEEN THE PLACEMENT OF SHOTBLOCKER AND THE INJECTION, COMPLETELY REMOVE SHOTBLOCKER FROM THE SKIN. REPEAT THE PROCESS BEGINNING WITH STEP 2.**
5. After you have completed the injection and withdrawn the needle, remove and clean ShotBlocker® with soap and water. Then place back in sealable bag with instructions for next use.

Questions? Call 1.419.727.8421

Try these other Bionix® Health at Home personal care products:

OtoClear® Ear Wax Removal
Igloo® Wound Irrigation

ShotBlocker® Injection Pain Reducer
Controlled Flow® Baby Feeder
SafeStraw™ Drinking Aid